

CompX eLock



Pre-set Supervisor Code

1, 2, 3, 4, 5, (#)
(Press in Sequence)

Pre-set User Code

1, 2, 3, 4, (#)
(Press in Sequence)

Notes:

- System is set automatically to re-lock in 5 minutes
- To lock the cart on demand – Press (*)
- If the system locks while a drawer is open; close the drawer, and it will lock automatically

Quick Programming Guide

ADD A USER:

1. Input a valid supervisor PIN and press (#)
2. Press and hold (#) until beeping stops
3. Input the new user PIN (4 digit min) and press (#)

ADD A SUPERVISOR:

1. Input a valid supervisor PIN and press (#)
2. Press and hold (#) until beeping stops
3. Press and hold (#) again until beeping stops
4. Input new supervisor PIN (4 digit min) and press (#)

DELETE A USER OR SUPERVISOR:

***Do NOT delete all supervisors or you will not be able to add or delete users or supervisors**

1. Input a valid supervisor PIN and press (#)
2. Press and hold (*) until beeping stops
3. Input the PIN to be deleted and press (#)

ADD A USER WITH A SPECIFIC SLOT ASSIGNMENT:

1. Input a valid supervisor PIN and press (#)
2. Press and hold (#) until beeping stops
3. Enter slot number that will be assigned to user and press (#)
(example: to program slot 6 – type 006 (#))
4. Input the new user PIN (4 digit Min) and press (#)

DELETE A USER WITH A SPECIFIC SLOT ASSIGNMENT:

1. Input a valid supervisor PIN and press (#)
2. Press and hold (*) until beeping stops
3. Enter slot number to be cleared and press (#)
(example: to clear slot 6 – type 006 (#))
4. Enter slot number to be cleared again and press (#)
(example: to clear slot 6 – type 006 (#))

Other Features

CompX eLock Override

Carts with the CompX eLock system manufactured by Specialty Carts come with a manual key override. Simply insert the appropriate key in the lock cylinder, located in the front of the cart, turn the key and pull outward approximately one inch. This unlocks all drawers until the cylinder is pushed back in to engage the lock system.



CompX eLock – Battery Replacement

Requires 6 ea “D” cell batteries

1. Use key override to unlock cart
2. Remove 1st large (10” tall drawer)
3. Battery holder is located on the interior of the right side of the cart
4. Remove old batteries and replace with 6 new “D” cell batteries in the same sequence



12949 Maurer Industrial Dr.

St. Louis, MO 63127

(314) 843-4407

(888) 588-0081